


Antigenski testovi za samotestiranje - za učenice i učenike

Samo 2 cm duboko u nosu okretati!
Kratko i bezbolno! Rezultat za 15 minuta!

Šta je za to potrebno?



testna kartica

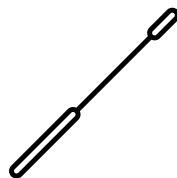



štapčić s tupferom



otopina/rastvor




Kako se izvodi?

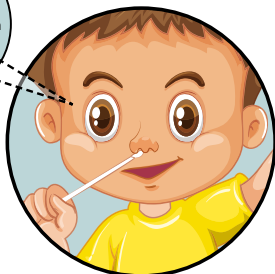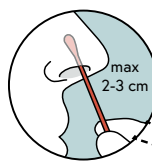


1. korak:

- Tupfer se stavlja SAMO 2 cm u nosnu šupljinu
- Okreće se 5 puta u krug
- Zatim se stavlja u drugu nosnicu i ponovo se okreće

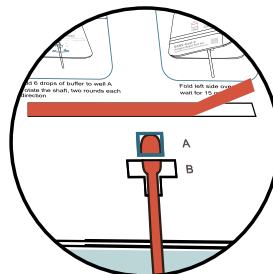


2. korak:

- Otklanja se zaštita s ljepljive trake i uzorak se stavlja u otvor

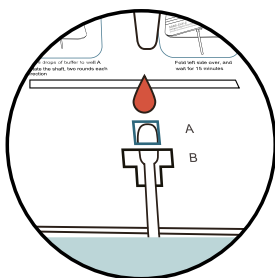

3. korak:

- Nakapa se 6 kapi otopine/rastvora u udubljenje i tupfer se okreće 2 puta u oba smjera

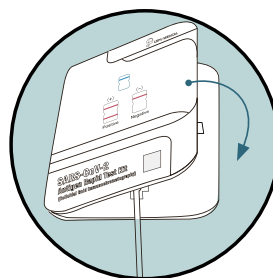

4. korak:

- Lijeva i desna strana kartice se zalijepe
- Rezultat testa pojavljuje se za 15 minuta



5. korak:

- Pozitivan rezultat testa:
kod kuće: kontaktirati broj 1450
u školi: uprava škole informiše 1450 ili lokalnu zdravstvenu službu
- Negativan test:
Ostani zdrav/a i posjećuj školu i dalje